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2022-10-10 LOT 221010
2027-10-09

MD

INSTRUCTIONS FOR SILICONE RESUSCITATOR BAGS

THIS DEVICE IS TO BE USED BY PROPERLY QUALIFIED C.P.R. TRAINED PERSONNEL

WARNING AND CAUTION

Test the device before use.

Never operate the resuscitator bag in toxic circumstances or in presence of gas.

In order to maintain its best performance, please use the product within 60 months from manufacturing date listed on the date lot label.

Do not administer supplemental oxygen in presence of fire, spark or open flame.

INSTRUCTIONS FOR USE

Please read carefully this manual before use the device.

The operator should be trained and qualified for the use of this manual resuscitator. Thus, the manufacturer and the distributor are not responsible for any improper use of the device.

Before use ensure that all parts listed in the pictures are included and properly mounted. Use a small balloon and simulate ventilation in order to verify the correct functioning. Please check if the balloons inflates and deflates properly. Facial mask must be properly placed in order to get an optimal ventilation.

Patient's airways need to be checked before ventilation. Should they be obstructed, ventilation activities will result unworthy. Thus, airways must always be kept free. In case of vomit, the upper valve should be cleared from all possible obstructions by pressing hard the compressive unit, at a distance from the patient. If this fails, take the valve off and wash it. If secretions are not yet removed, please properly dispose the device.

This device is provided with a special pop-off valve that is usually kept opened so that, should an obstruction occurred, the air produced by pressing the bag would led to deflate so to not compromise airways or proceeding into the stomach causing vomit or other disease.

Professionally trained personnel only are allowed to shut the valve and perform forced-ventilation.

In the back of the device two connectors are placed. The bigger is to be used for connecting the reservoir, while the smaller is for the oxygen tubing.

Please ensure that oxygen tube is well connected. Oxygen dispersion may result in compromising ventilation and causing saturation in closed environmental thus be dangerous for people.

Please perform rhythmic ventilation correctly and accordingly to usual norms.

ASSEMBLY AND DISASSEMBLY

The following activities need to be performed before each use for ordinary and extraordinary washing activities.

ONLY USE HANDS

Unscrew posterior part and collect the one that will be placed inside the device. Pull frontal patient's valve strongly onward. Take the whole valve and unscrew bigger connector ring nut. Unscrew pop-off valve paying attention not to let springs get loose inside. At this point, the device is completely disassembled.

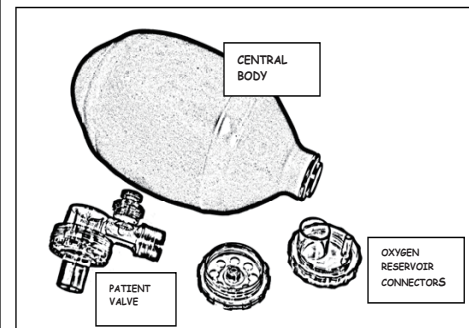
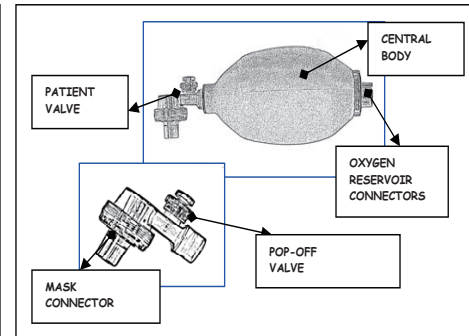
CLEANING AND MAINTENANCE

Dip all the parts into a hot water solution together with neutral soap or non-corrosive disinfectants.

Let parts immersed for some hours, then remove them from the solution and rinse in large amount of water. DO NOT ASSEMBLE THE DEVICE WHEN STILL WET..

Re-assembly activities should be carefully checked in order to preserve its perfect functioning. NEVER USE OILS ON FIXING RING NUTS.

ATTENTION: ALL THE PARTS OF THIS DEVICE ARE AUTOCLAVABLE AT 130° MAX.

Storage temperature	Dead space of patient valve
-20°C to 45°C	7.0 ml for all models
Operative temperature	pressure relief
-20°C to 45°C	40 cm H2O ± 5 cm H2O

connections:	
Patient port	15 mm I.D./ 22mm O.D
Reservoir connector	25mm I.D
Reservoir valve	26mm I.D (to bag inlet)
	25mm O.D (to oxygen reservoir)
Oxygen gas inlet	6 mm O.D

Ciclo ventilazione		volume pallone	
Model	Rate	Model	Volume
Adulto	45 bpm	Adulto	1630 ml

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Xiamen Winner Medical Co.,Ltd. China
Shanghai International Holding Corp.
GmbH(Europe)
PVS SpA - Cassina de Pecchi - I