

PRODUCT NAME

DISPO ICE SPRAY

PRODUCT CODE

SP400DISPONN - SP200DISPONN (NN: number of cans in the box)

PRODUCT DESCRIPTION

Ice spray. Cans content: 200 mL and 400 mL



TECHNICAL AND MATERIAL FEATURES

Ingredients: Isobutane, propane, butane, methylal, isopropyl myristate, menthol
LATEXFREE
The lot is indicated on the top of the can.
Expiration date: 5 years from the date of production. It is shown on the bottle cup.

CLASSIFICATION

Medical Device Class IIA
Classification according to Annex IX, paragraph III, rule 9 of European Directive 93/42/EEC and subsequent amendments (including Directive 2007/47/EC)

MANUFACTURER

Dispotech srl Via Al Piano 29 23020 Gordona (SO)
Tel. 0343.36711 Fax 0343.36567 e-mail: info@dispotech.it


INDICATIONS FOR USE/INSTRUCTION FOR USE

Used for relieving pain caused by injuries, scrapes, falls or sprains, and in cases requiring cryotherapy and application of cold. Thanks to its anaesthetic effect induced by cold on the injured area, sports activities can be resumed quickly. Shake before use. Spray product on the affected area from a distance of at least 20 cm for 3-4 seconds. Repeat application if necessary. The mechanical anaesthesia induced by cold eliminates pain temporarily.

STORAGE

Store at room temperature, away from heat, open flames or sparks.
Keep away from sunlight and do not expose to temperatures exceeding 50°C/122°F.

| | | | |
|---------------|----------|---------------|--|
| PCS/BOX | 12 or 24 | LAYERS PALLET | |
| CARTON SIZE | | PALLET SIZE | |
| CARTON LAYER | | PALLET WEIGHT | |
| CARTON PALLET | | | |

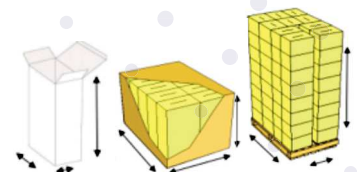

SYMBOLOLOGY

Symbology according to the norm UNI CEI EN ISO 15223-1:2017



REVISION DATE

01/06/2021 - REV. 06

APPROVED BY

DG

DISPOTECH srl

